

Alarms:

If unit alarms, read description on screen and call if instructed.

Chinook Respiratory Care



Chinook Respiratory Care

Simply Go Mini





Battery Life *Approximate*

| Flow | Breaths (20/min) |
|------|------------------|
| 1 | 12 hours |
| 2 | 9 hours |
| 3 | 6 hours |
| 4 | 4.5 |
| 5 | 4 hours |
| 6 | 3 hours |

Turning ON


- Press power button, press again to confirm ON.

Turning OFF

- Press power button, press again to confirm OFF.



Connect nose piece here

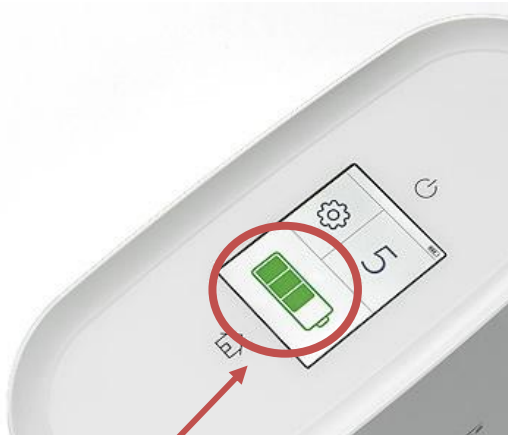

Symbol shows battery with each bar representing 25% of battery life.

Reminders:

- Always keep it fully charged when not using.
- When traveling keep it plugged into the car to maintain the charge. It will not fill the battery in the car, only maintain the amount of battery you already have.

Contact Us

Chinook Respiratory Care

255 12 'C' Street North
Lethbridge, AB T1H 2M6

403-329-9153

1-866-315-2954