

Alarms

NB: No Breath

This occurs when no breath is detected after a 2 minute period.

MX: Max Breath

High rate of breathing detected, alarm will stop when you are taking fewer breaths.

Any other beeping indicates an alarm or fault. Please see which symbols are showing on screen, and then call us.



Chinook Respiratory Care

255 12 'C' Street North

Lethbridge, AB

T1H 2M6

403-329-9153



Chinook Respiratory Care


Simply Go





Simply Go Portable Concentrator


Modes

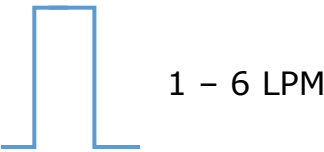

Pulse Mode: Always use this setting unless sleeping. Gives pulses on inspiration.



Continuous Flow: Use this when you go to sleep. (Meant for travel when you do not have

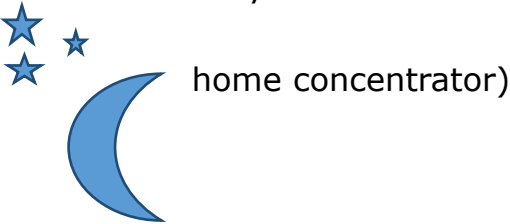

1 - 6 LPM

Sleep Mode: Use only if you need more than 2 LPM at night.

Turning ON

- Press the power button
- Confirm ON by pressing button a second time. **Pulse**

Turning OFF

- Press power button
- Confirm OFF by pressing button a second time.

Adjust Flow

- Press + or - symbols
- Note that the numbers go up or down in .5 increments

Battery Life (approximate)

Flow	20 Breaths/min
1	3.4 hours
2	3 hours
3	2.2 hours
4	1.7 hours
5	1.3 hours
6	1.3 hours
0.5 cont	2.9 hours
1 cont	1.6 hours
2 cont	0.9 hours

